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## ENG ANKLE SUPPORT

Please read the following instructions and keep for future reference.

The **ACTIVE ELASTICATED** range is made from a high-quality fabric, with the combined benefits of nylon, polyester as well as latex for superior strength, flexibility and comfort, to ensure a secure fit. The support is firm and effective on weakened muscles and joints, and helps prevent injuries. Each product is designed to give maximum support whilst maintaining a comfortable fit.

The Kedley Active Elasticated Range is suitable for both sporting and everyday use.

**Note:** The support should be sized correctly to ensure maximum benefit and to avoid over compression. If the support is being used following an injury, seek medical advice if no improvement in the condition is seen within 72 hours. If the injury is severe or if in any doubt seek medical advice immediately. If used by children, adult supervision is required. Keep out of reach of young children.

**Warning:** If soreness or a rash occurs discontinue use immediately and seek medical advice.

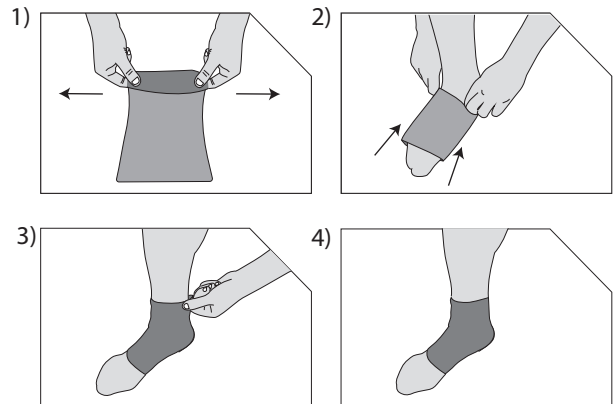
**Washing Instructions:** Hand wash in luke warm soapy water and air dry.

## ACTIVE ELASTICATED RANGE ANKLE SUPPORT

**Product code:** KED004/005/006/007

**Website:** [www.kedley.co.uk](http://www.kedley.co.uk)

**Address:** Braitrim House,  
98 Victoria Road,  
London, UK,  
NW10 6NB.

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